


Rapid Response™

Fentanyl (FYL) Test Strip

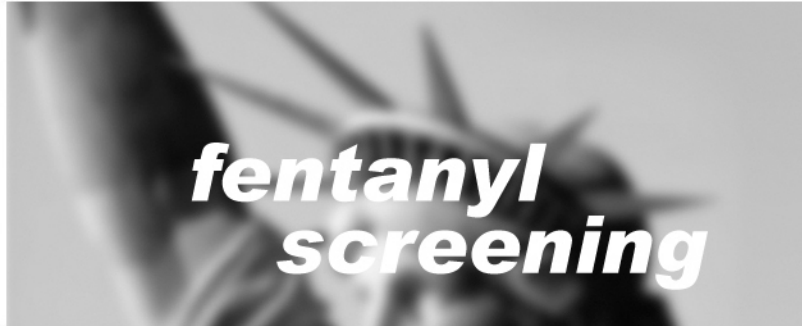

ONE LINE
FENTANYL

TWO LINES
NO FENTANYL



The Fentanyl Forensic Test Kit is a lateral flow Chromatographic Immunoassay for the qualitative detection of Fentanyl in liquid and powder substances at the cut-off concentration of 200 ng/mL.

Both are specific for Fentanyl screening with no significant cross reactivity to other opiates, such as Morphine and Heroin. The Rapid Response™ Fentanyl (FYL) Forensic Test Kit is able to detect Fentanyl and many other Fentanyl analogues such as Carfentanil, Acetyl Fentanyl, Butyryl Fentanyl, Remifentanyl, Ocfentanil, Sufentanil, p-Fluoro Fentanyl, Furanyl Fentanyl, Valeryl Fentanyl, and 3-Methyl Fentanyl.

Interpretation of test results are: positive (one line), negative (two lines), invalid (no lines or no control line). Test results are available in 5 minutes after test completion.

Test Strips

Product Information – Fentanyl 100 Strips

Product Code: LS_Fentanyl DOA-100
 Item #: FYL-1S48-100
 Sample: Liquid / Powder
 Format: Strip
 Quantity: 100 Tests/Kit
 Sensitivity: 20 ng/mL
 Specificity: Fentanyl
 Accuracy: > 98%
 Time-to-Result: 5 minutes
 Storage Condition: 2-30°C/36-86°F
 Test Principle: Lateral Flow Immunoassay
 Contents: 100 Test strips

Forensic Kit

Product Information – 5 Test Kit

Product Code: LS_FentanylIRRK-5
 Item #: FYL-18S7-5
 Sample: Liquid / Powder
 Format: Strip
 Quantity: 5 Tests/Kit
 Sensitivity: 200 ng/mL
 Specificity: Fentanyl
 Accuracy: > 98%
 Time-to-Result: 5 minutes
 Storage Condition: 2-30°C/36-86°F
 Test Principle: Lateral Flow Immunoassay
 Contents: Test strips, Swabs, Test Tubes, Extraction Buffers, Workstation, Product Insert

